

START UP PROCEDURES FOR THE BIODEX

- Check to ensure Main power switch is on. This may be left on if the Biodex is used often. If used daily, leave the main switch on. If not being used for a week or more the main switch may be turned off
- Power on the computer and dynamometer switches on the front panel
- Remove any attachments.
- If the main switch was OFF at start up the message “REMOVE ATTACHMENTS, PRESS START” will appear - Press Start on the control panel to initialize dynamometer
- Press START on the control panel
- If the main switch was ON at start up the message “DYNAMOMETER IN STANDBY” will appear - Press START on the control panel
- Turn on computer
- Turn on printer
- The system is now ready for use. The Setup Mode is automatically selected and the Select Limit button LED's should be flashing
- Ensure all the adjustment points (chair and dynamometer) are secure/tight
- Place the attachment to be used on the dynamometer – be sure to tighten the stopper fully
- Select mode from control panel
- Press Computer control. Use the System 3 software to enter subject, choose protocol, and set ROM
- Perform testing

***NOTE: As a safety precaution, the system will not function in any mode if the Remote Comfort Stop is not connected to the dynamometer.**

SHUT DOWN PROCEDURES FOR THE BIODEX

- Remove any attachments
- Be sure to exit and close the Biodex software
- Power down computer. Start>>Turn off computer>>Turn Off
- Turn off printer
- Power off the computer and dynamometer switches on the front panel
- The main switch may be left on if the Biodex will be used daily or within a week. If the down time is longer, turn main power switch on back of unit off

Technical Support: – supportservices@biodex.com or 1 800-224-6339

Biodex Website: – <http://www.biodex.com/>